

# OAKWORKS® PowerLine & Seychelle TREATMENT TABLE

## ASSEMBLY & OPERATION INSTRUCTIONS



H-Brace



Backrest Top



Removable Shelf



Backrest Top  
with Shelf



# TREATMENT TABLE

## H-Brace and Shelf

### SUPPLIED PARTS:

- |                                  |                              |                        |
|----------------------------------|------------------------------|------------------------|
| 1 - Stretcher Beam               | 4 - Long Large Bolts         | 1 - Large Allen Wrench |
| 12 - Flat Washers (shiny finish) | 4 - Short Bolts              | 1 - Small Allen Wrench |
| 4 - Disc Washers (dull finish)   | 8 - Velcro™ Strips for shelf | 4 - Lock Nuts          |
| 1 - Open End Wrench              | 2 - 2"Velcro™ Strips         | 4 - barrel nuts        |

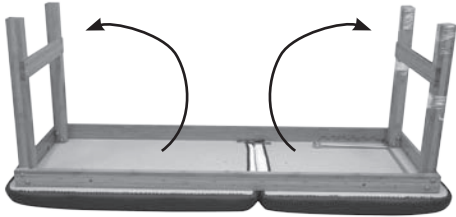
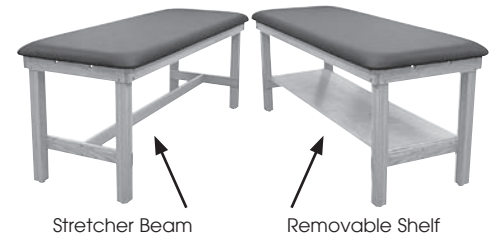


Fig. 1

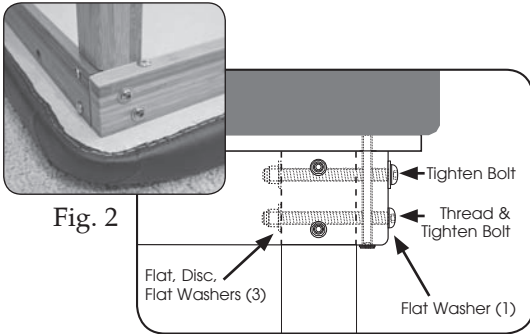


Fig. 2

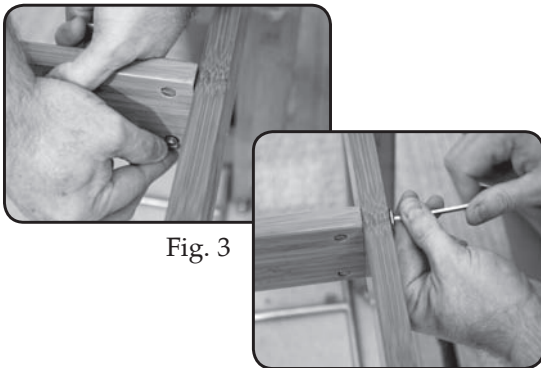


Fig. 3

### Assembly Instructions

1. Remove the shelf from the box (if ordered). Remove the table from the box and place it down on the floor top down. Remove all hardware and the stretcher beam.
2. Unfold the legs to a vertical position and remove any protective wrappings. (Fig. 1)
3. Install the Long Large Bolts, Washers and Nuts, as shown, on the top of each leg. When placing washers inside of the leg, sandwich 1 Disc Washer between 2 Flat Washers; outside of the leg, use 1 Flat Washer. Using the supplied tools, tighten every bolt -- 8 total -- as tight as possible. (Fig. 2)
4. Install the barrel nut into the stretcher beam being sure the screw slit is horizontal. Install the Short Bolts on either end of the stretcher beam and tighten the bolts on both ends. Make sure the stretcher beam is flush with the adjoining rails while tightening bolts. (Fig. 3)
5. Turn your table right side up. Get up on the table and rock back and forth to check that all bolts are tight. The table should be very stiff. If not, check and tighten the bolts as necessary with the supplied tools.

**NOTE:** *Retighten all bolts after 2 weeks; the wood will naturally experience slight shrinkage, which can cause the bolts to loosen.*

### Shelf Instructions

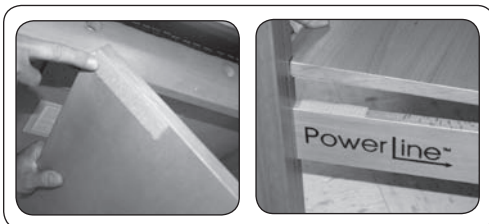


Fig. 4

If you have ordered a shelf, drop the shelf into place with the finished side up. The shelf has grooves at both ends that enable it to be held firmly in place by the legs. If necessary, the shelf can be further stabilized by installing the enclosed Velcro™ strips. Place strips on each corner of the shelf's bottom side, and on each end of the rails. Make sure not to mismatch Velcro™ strips; place all "hook" or all "loop" pieces on either the rail or shelf. (Fig. 4)

# TREATMENT TABLE

## Backrest Top Operation (if applicable)


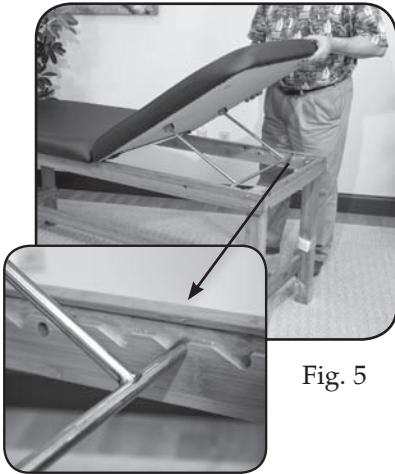


Fig. 5

### BACKREST TOP to raise the backrest

To raise the backrest, lift the table top at the head end of the table. Raise backrest to the desired height and release. Backrest support arm will lock in the nearest slot in the side rail. (Fig. 5)

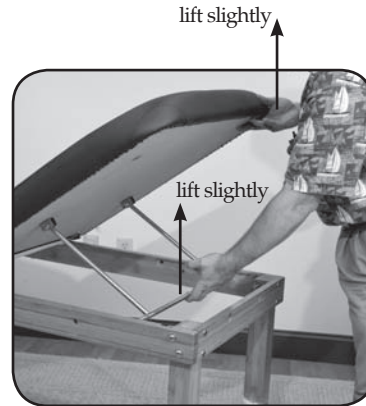

then lower backrest to the desired position.

Fig. 6

### BACKREST TOP to lower the backrest

To lower the backrest, lift the backrest slightly, lift and hold the support arm out of the slot and lower the backrest to the desired position. (Fig. 6)

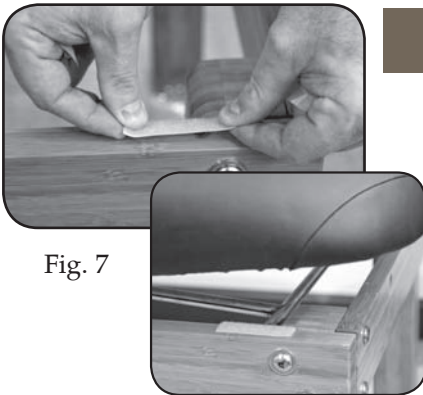



Fig. 7

### BACKREST TOP - Protect backrest top


Apply 2" Velcro™ strips over rail cut out. This will help protect your table top & provide a soft resting point for the backrest when the top is laid flat. (Fig. 7)

## PAPER ROLL HOLDER INSTALLATION

### Supplied Parts

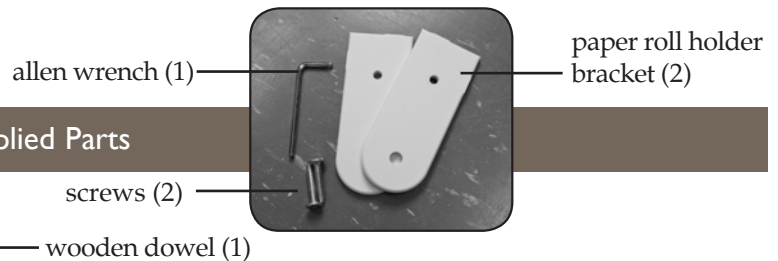


#### STEP 1

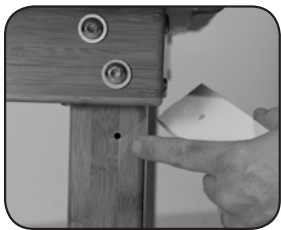


After the table is set-up, locate the hole in the leg.

#### STEP 2

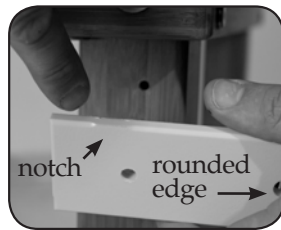


Locate one paper roll holder bracket. Be sure the notch is at the top & rounded edge away from the table.

#### STEP 3

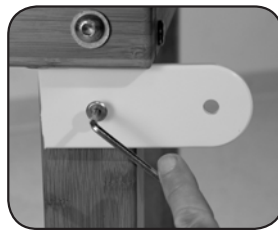

Using the supplied screw & allen wrench, secure the paper roll holder bracket into place. Repeat steps 1 thru 3 on the other side.

#### STEP 4

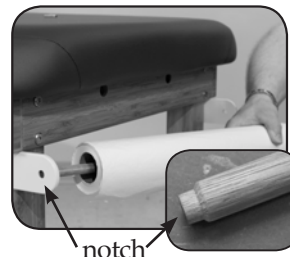

Locate the wooden dowel & feed it thru your paper roll. Place the notch on the wooden dowel in the hole located in the paper roll holder bracket.

#### STEP 5



To secure the wooden dowel on the other side, pull out the paper roll holder bracket slightly & insert the notch into the hole.

# Treatment Table Optional Accessories & Table Care

## Attaching the Table Extender:

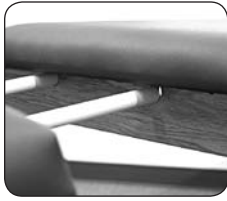


Fig. 8



Fig. 9

1. The table is equipped with two outlet holes at either end to allow the use of a Table Extender. Unfold the aluminum dowels of the extender and insert them into the outlet holes at either end of the table. (Fig. 8 & Fig. 9)

**CAUTION:** DO NOT sit, lean or use the Extender or Paper Roll Holder in mounting or dismounting from the table.

## Attaching & Using the QuickLock™ Face Rest:



Fig. 10

1. Insert the ends of the Face Rest frame into the outlet holes at either end of the table. (Fig. 10)
2. Starting with the face rest in a flat position, have your client lie face down on the face rest. Ask your client to support the face rest with two hands by holding onto it at the velcro on the sides of the U-shaped tube after you have unlocked the cam locks. (Fig. 11)
3. Have your client pull the face rest towards him/her, raising the chin and lowering the forehead until he/she is comfortable on the pad; cheeks fully supported and neck comfortably flexed. (Fig. 12)
4. When the face rest is in a position that meets your treatment requirements and your client's optimal comfort, lock the cam locks. (Fig.13)

**CAUTION:** DO NOT extend the dowels of the Face Rest further than 3" from the table portals to ensure a stable connection. (Fig. 14) The maximum weight capacity for the QuickLock™ Face Rest is 25 lbs. DO NOT sit, lean or use the face rest in mounting or dismounting from the table.

## Cleaning & Maintenance of the Treatment table:



Fig. 11



Fig. 12



Fig. 13

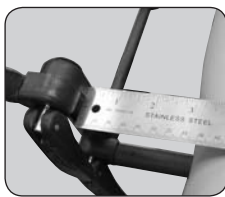


Fig. 14

- Clean your upholstered items only when needed with a mild detergent or Green Windex. A diluted 1/100 bleach solution for disinfecting can be used. Dry the table immediately after cleaning to remove excess cleaning solution. Using strong cleaners, such as alcohol, acetone, higher concentrations of bleach or other products that contain high concentrations of these substances, can shorten the life of your fabric, or discolor it. *Note: Damage caused by these cleansers will not be covered under the warranty.*
- Avoid extremes in heat. Very cold temperatures can make your fabric prone to cracking. Very high temperatures can cause the fabric to stretch permanently.
- Avoid exposing your products to direct sunlight for extended periods of time. This can lead to fading and cracking of the fabric.
- The wooden cabinetry of the equipment can be polished with any polish approved for wooden surfaces. Spray, or pour, the polish onto a soft cloth and be careful not to get it onto other surfaces.